



Service Configuration Online Help

All rights reserved. No part of this manual may be reproduced or copied in any form by any mean graphic, electronic, or mechanical, including photocopying, typing, or information retrieval systems without written permission.

Contents

Service Configuration Online Help

- Body Part/Position Mapping 2
- Service Configuration Overview 3
 - Starting Service Configuration 3
 - Scope of the Service Configuration 3
 - Buttons and Icons used in Service Configuration 4
 - Exporting Configuration Settings 5
 - Importing Configuration Settings 5
 - Comparing Configuration Settings 6
- DICOM Settings 7
 - DICOM Settings 7
 - Updating DICOM Settings 7
 - Modifying DICOM Settings 7
- HL7 Setting 9
- Modality Register 10
 - Registering a New Modality 10
 - Modifying a Registered Modality 11
 - Copying a Registered Modality 12
 - Deleting a Modality 13
- Modality Settings 14
 - Adding a New Modality 14
 - Changing a Modality Setting 15
 - Deleting Modality Settings 15
- Modality Worklist 17
 - Enabling the Modality Worklist Query 17
 - Disabling the Worklist Query 18
- Other PACS Registration 19
 - Registering a New PACS 19
 - Modifying a Third Party PACS Registration 20
 - Deleting a Third-Party PACS Registration 21
- Printer Register 22
 - Registering a New Printer 22
 - Modifying a Registered Printer 23
 - Adding a OEM DICOM Printer 24
- SMS Settings 25
 - Updating SMS Settings 25
 - Modifying SMS Settings 26
- Storage Settings 27
 - Updating Storage Settings 27
 - Modifying Storage Settings 28
 - Deleting Storage Settings 28

Service Configuration Online Help




Body Part/Position Mapping

Follow these steps to define the mapping rules between the procedure code and the body part/view position.

1. Click **Body Part/Position Mapping**.
2. Click **Add** .

Note

For Veterinary application, you must choose a species first.

3. Enter the name of the **Procedure Code**.
4. Select an option from the **Body Part** drop-down menu.
5. Select the desired **View Position** from the drop-down menu.
6. Click **OK**.
7. Click **Save** .
8. Click **Upload**.
9. Click **OK**.
10. Browse to the **Service** drive and select the **Backup** directory.
11. Click **ExportCfg**.
12. Browse to **Service > Backup**.
13. Name the **.cfg** file.
14. Click **Save** .
15. Click **Exit**  to close the **Service Configuration** window.
16. Access the Image Suite Utility Center and restart services.
You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Service Configuration Overview

Starting Service Configuration

This task must be performed by Service personnel only.

Follow these steps to get started on configuring the Service settings for Image Suite.

1. At the desktop of the Image Suite computer, double-click the **Image Suite Service Configuration** icon.

Image Suite software installs a shortcut on the desktop to access the Service Configuration screen.

If the shortcut is not available on the desktop, the Service Configuration screen can be accessed at **C > Program Files > Image Suite > PrimaryWS > Service_Cfg.exe**.

2. If necessary, make configuration updates.
3. Exit **Service Configuration** screen and restart all services to allow configuration updates to become effective.



Important

An error is indicated by messages with red text. The error message displays in the system log bar. If you get an error, you must check the configuration or contact Carestream Health Service.

Scope of the Service Configuration

The following information applies to Service personnel only.

The configuration dialog includes eleven configuration menus.

The left side of the dialog lists the menus, including:

Menu	Description
Printer Register	Add, remove, or update a DICOM printer.
Modality Register	Add, remove, or update a DICOM modality.
Other PACS Registration	Add, remove, or update a third-party PACS.
Storage Settings	Update any storage setting for CARESTREAM Image Suite.
System Information	View system setting information.
DICOM Settings	View DICOM settings.
SMS Settings	Update settings for system storage management. Add configuration for the Image Archive and Storage LRU tasks.






Service Configuration Overview



Menu	Description
Modality Settings	Maintain the list of supported modalities.
Body Part/Position Mapping	Define the mapping rules between the procedure code and body part/view position
Modality Worklist	Register modality worklist SCP server to provide the Modality Worklist for reconciliation. Enables CARE-STREAM Image Suite to act as the Modality Worklist SCU and support querying the Modality Worklist from the RIS.
HL7 Setting	Configure the remote EMR/EPR/PM IP address, port number, and code page.

To avoid configuration errors, configure the items sequentially as they are listed in the configuration dialog, starting with Printer Register and ending with HL7 Setting.

Buttons and Icons used in Service Configuration

The Configuration menus include one or more of the following buttons.


But- ton/Icon	Name	Function
	Add	Provides controls or a window to create a new configuration item or setting
	Delete	Removes a configuration item.
	Save	Retains configuration item settings.
	Import From File	Restores or overwrites the existing configuration settings with exported backup configuration settings.
	Copy Selected As	Copies the configuration settings

But- ton/Icon	Name	Function
	Edit	Provides controls or a window to modify the configuration item or setting
	Help	Links to the Service Configuration online help instructions for Image Suite

Exporting Configuration Settings

This task must be performed by Service personnel only.

Exporting the configuration settings allows you to save the settings to the Service directory in the configuration file **Lcpconfig.dat**.

1. Create a configuration setting in **Service Configuration** for a menu, such as **Printer Register** or **Modality Register**.
2. Click **Export Cfg**.
3. Browse to the **Service** directory.
4. Name the configuration file **systemconfig.dat**.
5. Click **Save** .

Importing Configuration Settings

This task must be performed by Service personnel only.


Importing the configuration allows you to overwrite the settings for a **Service Configuration** menu or restore a deleted configuration.

1. Select a menu in **Service Configuration**, such as **Printer Register** or **Modality Register**.
2. Click **Import Cfg**.
3. Browse to the **Service** directory.
4. Select the configuration file.
5. Click **Open**. The settings are applied to the menu.



Important

You must save and upload the configuration settings to retain the settings in the menu and make the settings active in the system.

6. Click **Save** .
7. Click **Upload**.

The settings are applied to the system.

Comparing Configuration Settings

This task must be performed by Service personnel only.

Comparing the configuration allows you to view a summary of the differences between an older configuration file and a newer configuration file.

1. Click **Compare Cfg**.
2. In **Select Old Configuration File**, browse to the **Service > Backup** folder and select the configuration file.
3. Click **Open**.
4. In **Select New Configuration File**, browse to the **Service Backup** folder and select the configuration file.
5. Click **Open**.
The utility displays a summary in a NOTEPAD file.
6. Review the data.
7. Save the .txt file (optional).

DICOM Settings

DICOM Settings

This topic covers the following tasks:

- Updating DICOM Settings
- Modifying DICOM Settings

Updating DICOM Settings

1. Click **DICOM Settings**.
2. For the DICOM settings, the required information is ImageSuite with no space and is case sensitive.
3. Enter data in the following required fields:

Required Field	Required Information
SSCP AE*	ImageSuite
SSCU AE*	ImageSuite
PSCU AE*	ImageSuite
MWLSCU AE*	ImageSuite
PortNo.*	ImageSuite


4. Click **Save** .

Modifying DICOM Settings



Important

For special integration cases; otherwise, do not change DICOM settings.

1. Click **DICOM Settings**.
2. Modify any of the required fields. For details about specific fields, see *Updating DICOM Settings*.
3. Click **Save** .
4. Click **Upload**.
5. In the **Modify Reason** box, enter a description.
6. Click **OK**.
7. Export the modified configuration. See *Exporting Configuration Settings*.



Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

HL7 Setting

This task must be performed by Service personnel only.

Follow these steps to define the interface that is used for communicating to the third-party EMR/EPR/PM systems.




Important

- You must obtain the HL7 specifications for the third-party EMR/EPR/PM systems from the customer.
- You must obtain the Image Suite HL7 specifications from the Carestream Health Service Portal and provide specifications to the customer.
- For all Image Suite HL7 interfaces with the third-party system, you must use the HL7 simulator for configuration and mapping.
- For detailed information on the HL7 Simulator, see the *Service Manual for the HL7 Simulator* on the Carestream Health Service Portal.
- The **Port No.** is assigned by the Image Suite System.

1. Under **HL7 Sender**, type the correct setting for:

Field	Information
IP Address	Third-party EMR/EPR/PM system.
Port No	Third-party EMR/EPR/PM system.
Code Page	Select the correct country from the drop-down list.

2. Click **Save** .
3. Optionally, click one of the following:
 - » **Export Cfg** to export settings to an external storage device.
 - » **Import Cfg** to import settings from an external storage device.
 - » **Compare Cfg** to compare an old configuration to a new configuration.
4. After saving, click **Upload** before exiting.

Modality Register

Registering a New Modality

1. Click **Modality Register**.
2. Click **Add** .



Important

- You must complete the required fields. Required fields are indicated with the asterisk symbol *.
- Changes to the modality registration are not valid until the SSCP Server restarts.
- You must enter a unique, exclusive name in the **ModalityName*** field.



3. Enter data in the following required fields:

Required Field	Required Information
ModalityName*	A unique name for the modality. The name must match the Station Name of the modality. The name is used for DICOMinput statistics.
AETitle*	The corresponding DICOM AE Title of the modality.
IPAddress*	The IP address of the modality.
ModalityType*	The type of modality.

4. Enter data in the remaining fields (optional).

Optional Field	Information
AEDescription	Any description of the modality.
PortNo	The DICOM port number of the modality. If the registered modality supports Storage Commitment SCU, specify the Port No and CARESTREAM Image Suite will send an N-EVENT-REPORT message to the modality through a new association on this port.
Manufacturer	The manufacturer of the modality.
ModelNumber	The model number for the modality.
Location	The location of the modality.

Optional Field	Information
SupportStorageCommitment	The DICOM storage commitment service confirms that an image is permanently stored by a device. The Service Class User modality uses the confirmation from the Service Class Provider (SCP) to confirm that it is safe to delete the images locally.
SC timeout(SEC)	The default value is 0 and indicates that the modality supports storage commitment. If the modality does not support storage commitment, assign a valued integer (seconds).
TransCode	Select if the modality is GB18030 or Japanese character set encoded.

5. Click **Save** .
6. Click **Reconciliation Rule**.
7. Select options based on customer-defined parameters.
8. Click **Save**.
9. Click **Exit**  to return to **Modality Register**.
10. Click **Upload**.
11. In the **Modify Reason** box, enter a description.
12. Click **OK**.
13. Click **ExportCfg**.
14. Browse to the **Service > Backup** directory.
15. Name the **.cfg** file.
16. Click **Save**.
17. Click the next menu (**Other PACS Registration**) to continue with configuration tasks.






Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Modifying a Registered Modality

1. Click **Modality Register**.
2. Click the name of the modality from the **NameList** menu.


3. Click **Edit**  to open the **Primary** window for the selected modality.
4. Modify any of the required or optional fields. For details about specific fields, see *Registering a New Modality*.
5. Click **Save** .
6. Click **Reconciliation Rule**.
7. Select options based on customer-defined parameters.
8. Click **Save**.
9. Click **Exit**  to return to **Modality Register**.
10. Click **Upload**.
11. In the **Modify Reason** box, enter a description.
12. Click **OK**.
13. Export the modified configuration. See *Exporting Configuration Settings*.



Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.



Copying a Registered Modality

1. Click **Modality Register**.
2. Click the name of the modality from the **NameList** menu.
3. Click **Copy Selected As**  to create a duplicate of the selected modality.



Important

You must enter a unique Net AE Name for the modality.


4. In the **Input Net AE Name** box, enter a unique name.
5. Click **OK**.
6. Modify any of the required or optional fields. For details about specific fields, see *Registering a New Modality*.
7. Click **Save** .
8. Click **Reconciliation Rule**.
9. Select options based on customer-defined parameters.
10. Click **Save**.
11. Click **Exit**  to return to **Modality Register**.
12. Click **Upload**.
13. In the **Modify Reason** box, enter a description.

14. Click **OK**.
15. Export the modified configuration. See *Exporting Configuration Settings*.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Deleting a Modality

1. Click **Modality Register**.
2. Click the name of the modality from the **NameList** menu.
3. Click **Delete** .

**Important**

Clicking **Yes** in the **Service.cfg** prompt removes the configuration.

4. Click **Yes** at the **Service.cfg** prompt. The configuration is immediately removed from the system.

**Note**

If you exported .cfg files, you can restore the modality configuration using **ImportCfg**. See *Importing Configuration Settings*.

5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Export the modified configuration. See *Exporting Configuration Settings*.

Modality Settings

Adding a New Modality

This task must be performed by Service Personnel only.

1. Click **Modality Settings**.
2. Click **Add** .



Important


- An SOP class must be in accordance with the DICOM standard.
- You must complete the required fields. Required fields are indicated with the asterisk symbol *
- You must restart the system services after updating the settings. The Modality Settings remain inactive until CARESTREAM Image Suite services are restarted.
- The default SOP is MG.

3. Enter data in the following required fields:

Required Field	Required Information
Modality Name*	A name for the modality.
SOP Class*	See the conformance statement.

4. Enter data in the remaining fields (optional).

Optional Field	Information
Alias	This field is normally empty.
Compression	Select No Compression or Lossless Compression.

5. Click **OK**.
6. Click **Upload**.
7. In the **Modify Reason** box, enter a description.
8. Click **OK**.
9. Click **ExportCfg**.
10. Browse to **Service > Backup**
11. Name the **.cfg** file.
12. Click **Save** . Click the next menu (**Modality Worklist**) to continue with configuration tasks.


**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Changing a Modality Setting

This task must be performed by Service personnel only.

**Important**


- By default, CARESTREAM Image Suite supports up to ten (10) modalities. You can change the default modalities.
 - You must restart the system services after updating the settings. The Modality Settings remain inactive until CARESTREAM Image Suite services are restarted.
1. Click **Modality Setting**.
 2. Select an existing modality and click **Edit** .
 3. Modify any of the required or optional fields. For details about specific fields, see *Adding a New Modality*.
 4. Click **OK**.
 5. Click **Upload**
 6. In the **Modify Reason** box, enter a description.
 7. Click **OK**.
 8. Export the modified configuration. See *Exporting Configuration Settings*.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Deleting Modality Settings

This task must be performed by Service personnel only.

1. Click **Modality Settings**.
2. Click the name of the modality from the **Modality Name List** menu.
3. Click **Delete** .



Important

Clicking **Yes** in the **Service.cfg** box removes the configuration.

4. Click **Yes** in the **Service.cfg** box.



Note

If you exported **.cfg** files, you can restore a modality configuration using **ImportCfg**. See *Importing Configuration Settings*.

5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Export the modified configuration. See *Exporting Configuration Settings*.

Modality Worklist

Enabling the Modality Worklist Query

This task must be performed by Service personnel only.



Important

- You must select Connect 3rd Party RIS to activate the parameter fields.
- You must restart all the services for the parameters to take effect, including the MWL SCU server. The system will run the MWL SCU in the background and retrieve the Worklist periodically.

1. Click **Modality Worklist**.
2. Select **Connect 3rd Party RIS**.
3. Enter data in the following required fields:

Required Field	Required Information
Peer RIS AE Title*	The AE title of the RIS that will provide Modality Worklist SCP.
Station Name*	The name of the station that will be retrieved for the Modality Worklist.
IP Address*	The IP address of the server that provides the Modality Worklist SCP.
Port*	The port number opened and connected for communication.
Procedure Start Time*	The duration dating back to query and retrieve the Modality Worklist.


4. Enter data in the remaining fields (optional).

Optional Field	Information
ConnectPatientXML	If you select Connect PatientXML, the system reads and parses a specified xml file to get the patient/order information.

Optional Field	Information
QueryInterval(Sec)	The interval time in seconds to trigger the retrieving operation.
Auto Update fromRIS	Check if you want your RIS to do automatic updates.

Disabling the Worklist Query

This task must be performed by Service personnel only.

1. Click **Modality Worklist**.
2. Deselect Connect 3rd Party RIS.
3. Click **Save** .
4. Click **Upload**.
5. In the **Modify Reason** box, enter a description.
6. Click **OK**.
7. Export the modified configuration. See *Exporting Configuration Settings*.



Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Other PACS Registration

Registering a New PACS

1. Click **Other PACS Register**.
2. Click **Add** .



Important

- You must complete the required fields. Required fields are indicated with the asterisk symbol *.
- You must enter a unique name in the AENAME* field.

3. Enter data in the following required fields:

Required Field	Required Information
AENAME*	A unique name for the third-party PACS. The name will be displayed in the source or destination list box of relative panels for CARE-STREAM Image Suite.
IPAddress*	The IP address of the third-party PACS.





Note

- Some of the optional DICOM-related settings are inactive. Selecting the checkbox will activate the setting for configuration.
- Identical SSCP AE Titles for different PACS systems may cause image retrieval to fail.
- You must use a different SSCP AE Title for each separate third-party PACS.
- The AE Title is case sensitive.

4. Enter data in the remaining fields (optional).

Optional Field	Information
SSCP AE Title	The DICOM title of the third-party PACS.
SSCP Port No	The DICOM port number of the third-party PACS.
Storage Commitment	Select Not Allowed or Different Association .
SSCU AE Title	The DICOM title of the third-party PACS. Each PACS must have a unique title.

Optional Field	Information
Modality Type	The image type(s) that will be transmitted from the third-party PACS.
ModelNumber	The model number for the third-party PACS.
AEDescription	Any description of the third-party PACS.
Location	The location of the third-party PACS.
Manufacturer	The manufacturer of the third-party PACS.

5. Click **Save** .
6. Click **Exit**  to return to **Other PACS Register**.
7. Click **Upload**.
8. In the **Modify Reason** box, enter a description.
9. Click **OK**.
10. Click **ExportCfg**.
11. Browse to **Service > Backup** directory
12. Name the **.cfg** file.
13. Click **Save**.
14. Click the next menu (**Storage Settings**) to continue with configuration tasks.



Important



You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.


Modifying a Third Party PACS Registration



Important

Changes to a third-party PACS registration may remain inactive until specific services restart, such as SSCP Server or Rule Server.


1. Click **Other PACS Registration**.
2. Click the name of the PACS from the **NameList** menu.
3. Click **Edit**  to open the **Primary** window for the selected PACS.
4. Modify any of the required or optional fields. For details about specific fields, see the tables in *Registering a New PACS*.
5. Click **Save** .

6. Click **Reconciliation Rule**.
7. Select options based on customer-defined parameters.
8. Click **Save**.
9. Click **Exit**  to return to **Other PACS Registration**.
10. Click **Upload**.
11. In the **Modify Reason** box, enter a description.
12. Click **OK**.
13. Export the modified configuration. See *Exporting Configuration Settings*.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual

Deleting a Third-Party PACS Registration

1. Click **Other PACS Registration**.
2. Click the name of the PACS from the **NameList** menu.
3. Click **Delete** .

**Important**

Clicking **Yes** in the **Service.cfg** box removes the configuration.

4. Click **Yes** in the **Service.cfg** box.

**Note**

If you exported .cfg files, you can restore a PACS configuration using ImportCfg. See *Importing Configuration Settings*.

5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Export the modified configuration. See *Exporting Configuration Settings*.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Printer Register

Registering a New Printer

1. Click **Printer Register**.
2. Click **Add** .

 **Note**

- You must complete the required fields. Required fields are indicated with the asterisk symbol *.
 - You must enter a unique, exclusive name in the **PrinterName*** field.
3. Enter data in the following required fields:

Required Field	Required Information
PrinterName*	A unique name for the printer. The name will be displayed in the Print Panel for CARE-STREAM Image Suite.
AETitle*	The corresponding DICOM AE Title of the printer.
PortNo*	The DICOM port number of the printer.
IPAddress*	The IP address of the printer.
PageFormat*	The format for the number of pages printed to a sheet of paper (media).Note: Some modalities cannot support a special layout, such as 1x3 or 3x1. Refer to the printer manual for details.a. Select all the formats supported by the printer.b. Click to add the formats to the right list box.
MediaSize*	The physical dimensions of the sheet of paper (media).

 **Important**

- An unsupported printer model could affect the quality of the output. For optimum results, use a supported printer model.
- You must use a gray scale printer type.

4. Enter data in the remaining fields (optional).


Optional Field	Information
AEDescription	Any description of the printer.
Manufacturer	The manufacturer of the printer.
ModelNumber	The model number for a supported printer model.
MediaType	The type of media (film).
Location	The location of the printer.
DfMediaSize	The default print film size for the printer.
Printer Type	The type of printer. Select the gray scale option.

5. Click **Save**.
6. Click **Yes** at the **Service_cfg** box.
7. Click **Upload**.
8. In the **Modify Reason** box, enter a description.
9. Click **OK**.
10. Click **ExportCfg**.
11. Browse to the **Service > Backup** directory.
12. Name the **.cfg** file.
13. Click **Save**.
14. Click the next menu (**Modality Register**) to continue with configuration tasks.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Modifying a Registered Printer

1. Click **Printer Register**.
2. Select the name of the printer from the **NameList** pull-down menu.
3. Modify any of the required or optional fields. For details about specific fields, see the tables in *Registering a New Printer*.
4. Click **Save** .


5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Export the modified configuration. See *Exporting Configuration Settings*.



Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Adding a OEM DICOM Printer

1. Click **Printer Register**.
2. Click **Edit** .
3. Select the model of the printer from the **Model list** pull-down menu.
4. Use the **Add Model** button to add a printer model into the model list.
5. Type all of the required or optional information into the fields.
6. Click **Save**.
7. Click **Upload**.
8. In the **Modify Reason** box, enter a description.
9. Click **OK**.

SMS Settings

SMS settings control System Storage Management processes such as archiving, backup, and recovery.

Updating SMS Settings

This task must be performed by Service personnel only.



Important

- You must complete the required fields. Required fields are indicated with the asterisk symbol *.
- You must restart the system services after updating the settings. The SMS Settings remain inactive until CARESTREAM Image Suite services are restarted.

- Click **SMS Settings**.
- If the image is not to be backed up, select **None Archive Mode**.




Note

The Pre-CD buffer is used to archive and backup image data and enable image data recovery in case of image loss during online storage. If a new device to store more online images is needed, additional storage can be added through the Service Configuration Tool.

- Enter the following data for the preferred Image PreCD and the Database PRECD.


Required Field	Required Information
Disk Count	The number of disk data buffers stored before writing to disk or media.
Prefix	Separates the three PreCD types by prefix.
Disk Type	The type of media such as CD or DVD.
Disk Size	The size of each buffer.
Location	The temporary storage directory for the buffer prior to writing to disk or media.

- Under **Image Archive** and **Storage LRU**, type the correct setting for:
 - Interval
 - Time Slice
- Click **Save** .
- Click **Upload**.
- In the **Modify Reason** box, enter a description.

8. Click **OK**.
9. Click **ExportCfg**.
10. Browse to **Service > Backup** directory.
11. Name the **.cfg** file.
12. Click **Save**.
13. Click the next menu (**Modality Settings**) to continue with configuration tasks.
You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity*.

Modifying SMS Settings

This task must be performed by Service personnel only.

1. Click **SMS Settings**.
2. Modify any of the required fields. For details about specific fields, see the table in *Updating SMS Settings*.
3. Click **Save** .
4. Click **Upload**.
5. In the **Modify Reason** box, enter a description.
6. Click **OK**.
7. Export the modified configuration. See *Exporting Configuration Settings*.

Storage Settings

Updating Storage Settings

This task must be performed by Service personnel only.

1. Click **Storage Setting**.
2. Click **Add** .



Important

- You must complete the required fields. Required fields are indicated with the asterisk symbol *.
 - You must restart the system after updating the settings. The storage settings remain inactive until CARESTREAM Image Suite is restarted.
 - For optimum performance, use the local storage system for storing image data.
3. Enter data in the following required fields:

Required Field	Required Information
AENName(A)*	The description for the storage.
RootDir(R)*	The directory to access on the StorageAE, such as Image or cachelImage.
Capacity(C)*	The maximum allocable capacity of this StorageAE.




Important

- It is recommended that the FE check and correct this value periodically.
- If this value is not checked or corrected it can cause a waste of storage space.

Optional Field	Information
Information Used Space	This field will be updated by the system automatically.

4. Click **OK**.
5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Click **ExportCfg**.




9. Browse to the **Service > Backup** directory.
10. Name the **.cfg** file.
11. Click **Save** .
12. Click the next menu (**System Information**) to continue with configuration tasks.

Note

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Modifying Storage Settings

This task must be performed by Service personnel only.


1. Click **Storage Settings**.
2. Click the name of the StorageAE from the **NameList** menu.
3. Click **Edit** .
4. Modify any of the required fields. For details about specific fields, see the table in *Updating Storage Settings*.
5. Click **Save** .
6. Click **Exit**  to return to **Storage Settings**.
7. Click **Upload**.
8. In the **Modify Reason** box, enter a description.
9. Click **OK**.
10. Export the modified configuration. See *Exporting Configuration Settings* in the Service Manual.

Important

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

Deleting Storage Settings

This task must be performed by Service personnel only.

1. Click **Storage Settings**.
2. Click the name of the StorageAE from the **NameList** menu.
3. Click **Delete** .

**Important**

Clicking **Yes** in the **Service.cfg** box removes the configuration.

4. Click **Yes** in the **Service.cfg** box.

**Note**

If you exported .cfg files, you can restore a storage configuration using **ImportCfg**. See *Importing Configuration Settings*.

5. Click **Upload**.
6. In the **Modify Reason** box, enter a description.
7. Click **OK**.
8. Export the modified configuration. See *Exporting Configuration Settings*.

**Important**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.

System Information

This task must be performed by Service personnel only.

 **Note**

CARESTREAM Image Suite can act as a service class provider and as a service class user.

1. Click **System Information** and review the information.
2. Click the next menu (**DICOM Settings**) to continue with configuration tasks.

 **Note**

You must use the Image Suite Utility Center to restart services and activate the new configuration. After all configuration tasks are complete, see *Controlling System Services and Connectivity* in the Service Manual.



Carestream Health

150 Verona Street

Rochester, NY 14608

United States

© Carestream Health, Inc., 2013

Printed Made in the United States.

CARESTREAM is a trademark of Carestream Health.

Pub No. AB1136_en

Rev A

